**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE
ユーザーマニュアル**

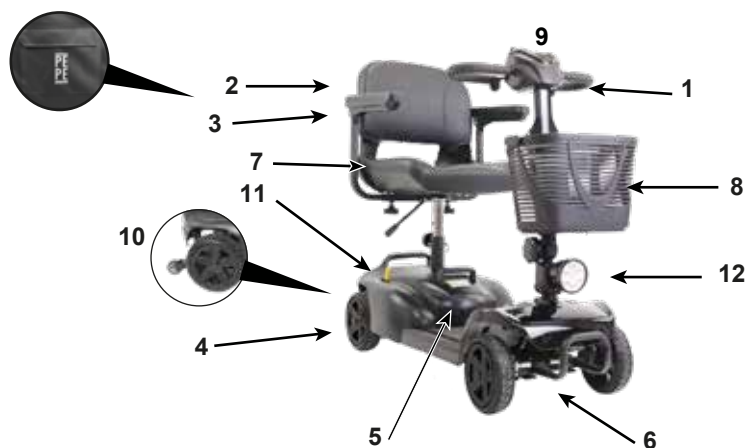
PEPE - SCOOTER

PEPE - SCOOTER (BLACK/NEGRO/SCHWARZ/NOIR/NERO/ブラック)

P10044

1 COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI / コンポーネント

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	ジャポネーズ
1.	Handlebar	Manillar	Lenker	Guidon	Manubrio	ハンドルバー
2.	Backrest	Respaldo	Rückenlehne	Dossier	Schienale	バックレスト
3.	Armrest	Reposabrazos	Armlehne	Accoudoir	Bracciolo	アームレスト
4.	Wheel	Rueda	Rad	Roue	Ruota	ホイール
5.	Battery	Batería	Batterie	Batterie	Batteria	バッテリー
6.	Anti-collision bar	Barra anti-colisión	Anti-Kollisionsstange	Barre anti-collision	Barra anticollisione	衝突防止バー
7.	Seat	Asiento	Sitz	Siège	Sedile	シート
8.	Basket	Cesta	Korb	Panier	Cestino	バスケット
9.	Controller	Controlador	Steuergerät	Contrôleur	Comando	コントローラー
10.	Anti-tip wheel	Rueda antivuelco	Anti-Kipp-Rad	Roue anti-bascule	Ruota antiribaltamento	転倒防止ホイール
11.	Manual brake lever	Palanca freno manual	Handbremshebel	Levier de frein manuel	Leva del freno manuale	手動ブレーキレバー
12.	Front light	Luz frontal	Vorderes Licht	Feu avant	Luce anteriore	フロントライト

2 DIMENSIONS / DIMENSIONES / ABMESSUNGEN / DIMENSIONS / DIMENSIONI / 寸法

WIDTH	HEIGHT	LENGTH
49 cm (19")	94 cm (37")	108 cm (42.5")

SEAT HEIGHT	SEAT DIMENSIONS	BACKREST HEIGHT	Ø WHEEL
64 cm - 53 cm (25.2" - 21")	45x43 cm (17.7" - 16.9")	39,5 cm (15.5")	20 cm (8")

MAX. USER WEIGHT	NET WEIGHT	WEIGHT WITH BATTERY
120 kg (265 lb)	40,5 kg (89.3 lb)	45 kg (99 lb)

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- EN** Fitting the seat bracket
1. Insert the seat bracket into the inner tube located on the rear body.
 2. Tighten the screws.
 3. Insert the seat in the outer tube.

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- EN**
1. Place the battery in the tray. Note that the electrode terminals of the battery must be aligned with those of the rear body.
 2. Check and clean the electrodes and remove any foreign bodies on them that may cause poor electrical contact.

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EN Mounting the basket
1. Mount the basket on the bracket.

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4

ADJUSTMENT/ AJUSTE / ANPASSUNG / AJUSTEMENT / REGOLAZIONE / ユーザーへの調整

EN Seat height (3 regulations)
1. Loosen the seat nut to release the seat.
2. Pull the seat upwards.
3. Remove the pin by pulling the ring outwards.
4. Adjust the seat height.
5. Refit the pin.
6. Tighten the seat nut.

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**EN** Adjusting the armrest

1. Width: Adjust the width between the armrests to suit you and tight the nut.
2. Angle of inclination: Raise the armrests, turn the bolt as you want more or less tilt and lower the armrests.

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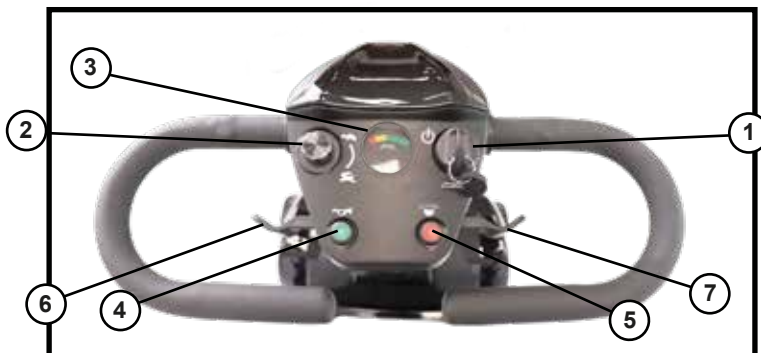
**EN**

- 1. Key switch:** Turns on and off the scooter.
- 2. Speed selector:** Set the maximum speed of the scooter, turn clockwise to increase speed. Turn it anticlockwise to decrease the speed.
- 3. Power indicator:** Lights up when you turn on the scooter with the key. The remaining battery capacity is indicated by 3 colours.
 - Green: Battery fully charged.
 - Yellow: Battery are half full and need to be recharged.
 - Red: Battery are fully discharged and should be recharged immediately.
- 4. Horn button:** Press this button to sound the horn.
- 5. Front light button:** Press this button to turn the front light on or off.
- 6. Left control lever:** Pull the lever towards yourself with your left hand, the scooter moves backwards. The greater the angle, the greater the speed. Slowly release the lever and the scooter will slow down. When it is completely released and the lever is readjusted, the scooter stops and is in a braking state.
- 7. Right control lever:** Pull the lever towards yourself with your right hand to move the scooter forward. The greater the angle, the greater the speed. Slowly release the lever and the scooter will slow down. When it is fully released and the lever is readjusted, the scooter will stop and will be in a braking state.

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- EN**
- Move the position of the yellow lever, this brake can be used to switch between “electric drive” and “hand pushing”.
- Electric drive: When the brake lever is moved back to the electric drive position, the scooter is in a braked state and cannot be pushed. In this position, switch on the power, so that the scooter can operate normally.
 - Hand pushing: When the brake lever is moved forward to the hand pushing position, the scooter is in a brake released state and can be pushed.
- Do not push the brake lever forward to the hand pushing position when going downhill, there is no braking effect at this time to ensure the safety of the user and the electric scooter. If the lever is in the hand pushing position, when starting the scooter it will beep, it is necessary to take out the key and turn it off.

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EN When reversing, pay attention to the surrounding conditions. Slowly pull the lever towards yourself with your left hand. When the left-hand lever is released, the scooter can come to a smooth stop.

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EN Materials: Aluminum tube, ABS.
Braking system for electric and manual driving.
Driving control unit.
Two keys for starting.
With storage basket with 10L capacity.
Reverse gear with beeps.
Handlebar tube without mirror extension.
Includes USB charger port.
Includes Allen key.
Black colour.
Supported weight: 265 lb.
Supported weight: 265 lb.
Braking distance ≈39".
Max. inclination of the driving surface: 9°.
Max. speed: 4.3 mph.

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9

WARNINGS / ADVERTENCIAS / WARNUNGEN / AVERTISSEMENTS / AVVERTENZE / 警告

- EN**
- Make sure you are familiar with the current legislation governing the use of the scooter in your area.
 - Incorrect battery installation may render the scooter inoperable.
 - Remove the ignition key before adjusting for user comfort, never do so while driving.
 - Your scooter's battery must be charged with the external battery charger supplied. Do not use any automotive type battery charger.
 - Disconnect the power supply before charging the battery.
 - DO NOT use the key switch to stop your scooter unless an urgent event has occurred.
 - If your scooter has been stationary for a long period of time, switch it off to prevent unintentional movement.
 - If your scooter moves unintentionally, release the throttle control lever immediately. The scooter will stop automatically unless this lever is out of service.
 - Before starting to use the scooter, set the speed adjustment knob to the lowest position.
 - When your scooter is in manual driving mode, the brake system is disabled and the throttle control lever functions are inhibited by the control system. The horn will sound while the power is on.
 - NEVER operate your scooter in manual drive mode without your supervisor. Failure to do so may result in personal injury.
 - NEVER put your scooter in manual drive mode on an incline.
 - DO NOT push the hand brake lever forward when the scooter is moving.
 - Incorrect connection of the charger can cause damage to the charger.
 - If the water on the road surface exceeds 5 cm, the motor may be permanently damaged. No water must get in.
 - Check all electrical connections. Make sure they are tight and not corroded.
 - Check the brakes and battery charge before each use.
 - Exceeding the weight limit voids your warranty and may result in personal injury and damage to your

scooter.

- When going up a slope, DO NOT zigzag or ride at an angle across the face of the slope, ride straight up the slope. This greatly reduces the possibility of a rollover or fall. Always use extreme caution when going up a slope.
- NEVER drive backwards up a slope.
- Driving up or down a slope steeper than 9° may cause your scooter to become unstable and cause it to tip over.

- Avoid driving on soft surfaces, tall grass, sand.

DO NOT use your scooter on slippery roads with ice or snow and in adverse weather conditions.

Your scooter may be affected by EMI/RFI. If unintended movement due to EMI/RFI occurs, immediately switch off your scooter and contact your dealer.

- To transfer to or from the scooter, make sure that it is not in manual driving mode.
- Avoid resting your full weight on the armrests and footrests, your scooter may tip over or be damaged.
- If lighted cigarettes are thrown on a padded seating system, it may cause a fire resulting in damage, serious injury or death.

- Occupants of the electric vehicle are at particular risk of death or serious injury from such fires and the gases they produce, as they may not be able to move away from the electric vehicle.

- DO NOT smoke while operating this electric vehicle.

- Improper supervision or maintenance could result in injury, damage, or death due to ingestion or suffocation caused by parts or materials.

Improper use of this product may result in injury or damage.

- If you have any questions regarding warnings, cautions, or instructions, contact a health care professional or your supplier before attempting to use this equipment.

- Turn off the electric vehicle before getting on, off, or handling unwieldy objects.

- NEVER leave the mobility device on a slope.

- If you must brake in an emergency, simply release the drive lever and allow the mobility device to come to a complete stop.

- DO NOT switch off the electric vehicle in motion unless there is no other option.

- NEVER transport the electric vehicle with the occupant seated in the vehicle.

- Avoid storing or using the electric vehicle near fire or combustible products.

- DO NOT exceed the maximum permissible load.

- Ensure that moving parts of the mobility device do not cause injury, especially when children or animals are nearby.

- The scooter is NOT recommended for use by minors.

- Before operation, familiarise yourself with the local pedestrian traffic regulations and practise operating the scooter. Make sure that the brakes are working properly, all functions are normal and the battery is fully charged before use.

- DO NOT operate under the influence of alcohol or drugs.

- The scooter is designed for one person only. DO NOT carry passengers or use for transporting or towing.

- Make sure you have all your belongings checked before you start and use the scooter.

- DO NOT put weight on the handlebars or armrests.

- DO NOT attach or tie a leash to the scooter to walk your pet.

- DO NOT insert your fingers between scooter parts. The use of gloves is recommended when handling scooter parts. Use extreme caution with metal and heavy parts.

- IMPROVING MOBILITY DEVELOPMENT SL. is not responsible for theft or loss of parts or the scooter.

- DO NOT expose the mobility scooter to direct sunlight for long periods of time. Metal surfaces and parts, such as the seat and armrests, could overheat.

- Do not connect any electrical device to the electric vehicle that has not been expressly certified by IMPROVING MOBILITY DEVELOPMENT SL. for this purpose.

- Improper use could cause the electric vehicle to smoke, spark or catch fire. This could result in damage, serious injury or death due to fire.

- DO NOT use the electric vehicle for any purpose other than that for which it was intended.

- If the electric vehicle starts to smoke, spark or burn, stop using it and have it repaired immediately.

- To avoid electric shock, check the plug and cord for cuts or frayed wires. Replace cut or frayed cords immediately.

- Corroded electrical components due to exposure to water or other liquids can cause damage, serious injury or death.

- Reduce exposure of electrical components to water or other liquids.

- Electrical components damaged by corrosion should be replaced immediately.

- Electrical vehicles that are frequently exposed to water or other liquids may require more frequent replacement of electrical components.

- Never cover the lighting system with textiles.

- A fault in the electrical system can lead to unusual behaviour, e.g. the light comes on steady, does not come on at all.
- Only climb gradients that do not exceed the maximum safety slope.
- Avoid braking sharply or accelerating on slopes.
- NEVER attempt to negotiate an obstacle when going up or down a slope.
- Always approach obstacles in a straight line. Make sure that the front and rear wheels go over the obstacle in one go, do not stop right in the middle.
- Avoid varying your centre of gravity as well as sudden changes of direction when the mobility device is in motion.
- Always distribute the weight evenly when carrying the mobility device. Always try to keep the centre of gravity of the electric vehicle in the middle and as close to the ground as possible.
- Drive in narrow spaces at minimum speed and with due caution.
- There is an increased risk of rollover when making turns.
- Reduce speed before making turns. Only accelerate when you have completed the turn.
- Note that the seat height has a major influence on the centre of gravity. The higher the seat height, the greater the risk of tipping over.
- Drive with extreme caution on soft surfaces, especially on uphill and downhill gradients.
- During the process, pay particular attention to the tipping stability of the electric vehicle.
- DO NOT attempt to carry out maintenance work that is not described in this user manual. Such repair and/or maintenance must be carried out by a qualified technician. Contact a dealer.
- For safety reasons and to prevent accidents resulting from inadvertent wear and tear, it is important that this electric vehicle is serviced annually under normal operating conditions.
- Under difficult operating conditions, e.g. daily driving on steep slopes or in the case of use in medical care with frequent changes of wheelchair users, it is advisable to carry out intermediate checks on the brakes, accessories and running gear.
- Spare parts must match the original parts.
- Always state the serial number of the electric vehicle to make it easier to order the correct spare parts.
- Only use batteries that have been approved by IMPROVING MOBILITY DEVELOPMENT SL. for this electric vehicle.
- Continued use of an electric vehicle that has not been adjusted to the correct specifications could cause erratic vehicle behaviour resulting in damage, serious injury or death.
- After making any adjustments, repairs or maintenance and before proceeding with use, make sure that all fastening parts are mounted and tightened securely.
- Ensure that the handgrip is correctly adjusted before riding the scooter.
- NEVER drive around obstacles at a steep angle.
- DO NOT attempt to get off the scooter on a slope or incline.
- Protect the battery charger from moisture. Always charge the battery in a dry environment.
- DO NOT use the battery charger if it has been dropped or damaged.
- Never attempt to recharge the battery by connecting the cables directly to the battery terminals.
- Only use an extension cable if absolutely necessary. If you must use an extension cable, make sure it is in perfect condition.
- Under no circumstances may this electric vehicle be used as a vehicle seat or to transport the user in a vehicle.
- Risk of twisting when lifting heavy parts. Use the correct lifting techniques.
- When cleaning electronic components, observe their protection class with regard to water ingress.
- Make sure that no water splashes onto the plug or wall socket.
- DO NOT touch the socket with wet hands.
- All cleaning agents and disinfectants used must be effective, compatible with each other and must protect the materials to be cleaned.
- Never use corrosive fluids (alkalis, acids, etc.) or abrasive cleaning agents.
- DO NOT use solvents (cellulose strippers, acetone, etc.) that change the structure of the plastic or dissolve adhered labels.
- Always make sure that the product has dried completely before using it again.
- The product must not be cleaned in automatic washing systems, using steam or pressure cleaning
- This product may contain substances that could be harmful to the environment if disposed of in sites (landfills) not suitable according to the legislation.
- DO NOT throw the battery into household waste.
- DO NOT throw the battery into the fire.
- The battery must be taken to a suitable clean point. There is no cost to deliver batteries and this is required by law.
- Only remove the uncharged battery.
- Cover the lithium battery terminals before throwing them away.

- Collisions between the components of the electric vehicle are possible due to the various combinations of adjustment options and their individual parameters. When adapting the seat system and its functions to the user, make sure that the components of the electric vehicle do not collide with each other.
- When adjusting the angle of the armrest, there may be some interlocking point, be careful with your fingers.
- Risk of serious injury if one of the armrests is removed from its support by being adjusted to a width greater than the permissible value.
- The CE marking is void if it is replaced or if components or accessories are added that have not been approved by IMPROVING MOBILITY DEVELOPMENT SL. for this product. In this case, the company adding or replacing the components or accessories will be responsible for the conformity assessment/CCE marking or registration of the electric vehicle as a special design and also for the related documentation.
- IMPROVING MOBILITY DEVELOPMENT SL. is not responsible for theft or loss of parts or the scooter.
- Not suitable for transport on aircraft.equipment.

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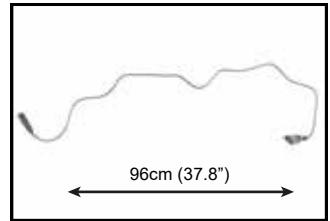
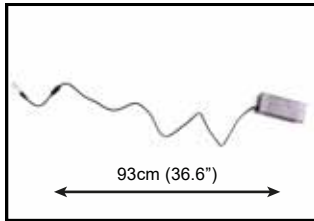
EN ■ The scooter's battery is charged with an external charger, which is included with the scooter. This charger consists of two parts, a cable with a plug for the light and a cable that connects directly to the scooter.

■ Turn off the scooter, lift the battery box cover and plug the external charger into the scooter's three-pin charger. While the batteries are charging, the scooter will not operate.

■ The charger red light indicates that the battery is charging correctly. When the light turns green, leave the scooter charging for another 1 to 2 hours. It is recommended to charge the battery for 10-12 hours.

■ The battery can be charged outside the scooter.

■ Unplug the scooter when the battery is charged.

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- EN** ■ Avoid knocking the handlebar console.
- Avoid prolonged exposure of your scooter to extreme conditions, such as overheating, cold or wet.
- Keep the handlebar console clean, wiping with a soft dry cloth is recommended.
- Check that all connectors are tight and secure.
- Check all electrical connectors, including the charger connectors, and make sure they are tight and not corroded. The battery should be placed flat in the battery tray with the battery terminals facing backwards and forwards towards each other and with the 3-pin charger socket facing backwards.
- When you finish daily use, remove the key to reduce unnecessary power consumption.
- This product is equipped with an energy saving system. When you stop using it for 20 minutes, it will automatically switch off. When you need it again, plug the key back in.
- You can apply a light coat of car wax to help maintain its high-gloss appearance.
- All wheel bearings are pre-lubricated and sealed. They do not require subsequent lubrication.
- Check the scooter before each use.

- Clean dirt from the surface of the motor, no water should get into it.
- Check that all wheels, bearings, handlebars, seat and other parts are in good condition.
- When the scooter is not in use, it is recommended that it be kept indoors or protected outdoors with a cover and properly supervised.

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13 WARRANTY / GARANTÍA / VERBÜRGUNG / GARANTIE / GARANZIA / 保証

- EN**
- IMPROVING MOBILITY DEVELOPMENT SL. warrants this device to be free from defects.
- Motor and controller: The warranty will remain valid for 2 years from the date of original purchase by the consumer.
 - Battery and charger: The warranty will remain valid for 1 year from the date of original purchase by the consumer.
 - The scooter (without motor and battery): The warranty will remain in force for 3 years from the date of original purchase by the consumer.
 - This warranty does not cover:
 - Replacement parts supplied by anyone other than IMPROVING MOBILITY DEVELOPMENT SL. or the authorised dealer.
 - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
 - Parts exposed to natural wear and tear that require periodic replacement.
 - IMPROVING MOBILITY DEVELOPMENT SL. is not liable for own damage, damage to third parties or damage to property, whether accidental or in bad faith.
 - IMPROVING MOBILITY DEVELOPMENT SL. is not responsible for theft or loss of parts or the scooter.

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EN |

Serial Number	Fault Phenomenon	Cause of Fault	Elimination Methods
1	The throttle lever is defective or the maximum speed is low	① Battery voltage is too low ② Control lever is released ③ Control lever is stuck	① Fully charge the battery ② Contact the supplier ③ Clean or replace the control lever
2	The motor does not work when turned on	① Battery voltage is too low ② Control lever wire is released ③ Motor failure	① Fully charge the battery ② Contact the supplier ③ Contact a professional or repair point
3	Insufficient mileage on a single charge	① Insufficient charging or charger failure ② Battery is aging or damaged ③ Many slopes, headwind, frequent braking, heavy load	① Fully charge or replace the charger ② Contact the supplier ③ Try to avoid these bad conditions
4	The charger does not work	① Charger plug is disconnected or connector is released ② Charger or scooter plug fuse is burned ③ Battery wires are disconnected	① Secure plugs and connectors ② Replace the fuse ③ Connect the wires
5	Abnormal noise during driving	① Screws or nuts are loose ② Wheel bearing are worn	① Tighten loose screws or nuts ② Replace the bearings
6	Power is interrupted during driving	① Controller protection actions ② Battery thermal protector activated ③ Motor overheating protection	① Turn the key off and back on ② Reset the thermal protector ③ Let the motor cool down before driving
7	Other faults	Other faults not covered in the guide above	Contact the supplier or repair station. Do not disassemble the scooter yourself

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READ BEFORE RETURNING, SAVE THE PLANET.

- EN** Returns generate more than 15 million tons of CO₂ annually.
- ES** Las devoluciones generan más de 15 millones de toneladas de CO₂ al año.
- DE** Retouren erzeugen mehr als 15 Millionen Tonnen CO₂ jährlich.
- IT** I resi generano oltre 15 milioni di tonnellate di CO₂ all'anno.
- FR** Les retours génèrent plus de 15 millions de tonnes de CO₂ par an.
- JA** リターンは年間1,500万トン以上のCO2を排出する。

- EN** If pieces are missing or have arrived damaged, before returning, **contact us**
- ES** Si faltan piezas o están en mal estado, antes de devolver **contacta** con nosotros.
- DE** Wenn Teile fehlen oder beschädigt sind, **kontaktieren** Sie uns vor der Rücksendung.
- IT** In caso di parti mancanti o danneggiate, prima della restituzione, **contattaci**.
- FR** Si des pièces sont manquantes ou endommagées, avant de retourner nous **contacter**.
- JA** 部品が不足していたり、状態が悪い場合は、返品する前に弊社までご連絡ください。

IMPROVING MOBILITY DEVELOPMENT S.L.
Avda. Mare Nostrum 1, Bloque 1, Pta 16
46120, Alboraya, Valencia, Spain
admin@kmina.com | www.kmina.com